# 3M

## Health Care Particulate Respirator and Surgical Mask 1860/1860S

### User Instructions

**IMPORTANT:** Keep these *User Instructions* for reference.

 <b>WARNING</b>
This respirator helps protect against certain particulate contaminants but does not eliminate exposure to or the risk of contracting any disease or infection. <b>Misuse may result in sickness or death.</b> For proper use, see supervisor, or <i>User Instructions</i> , or call 3M Health Care Helpline at 1-800-228-3957. In Canada, call 3M Helpline at 1-800-563-2921.

98-0060-0034-7\_2  
34-8710-9471-9

### IMPORTANT:

Before use, the wearer must read and understand these *User Instructions*. Keep these instructions for reference.

### Description:

The 3M™ Health Care Particulate Respirator and Surgical Mask 1860/1860S is designed to help provide respiratory protection for the wearer. This respirator has a filter efficiency level of at least 95% against particulate aerosols free of oil<sup>†</sup>. It is fluid resistant<sup>‡</sup> and meets > 99% bacterial filtration efficiency (BFE)<sup>±</sup>. It is disposable and may be worn in surgery. It can fit a wide range of face sizes.

 This product contains no components made from natural rubber latex.

### Intended Use:

This product meets CDC guidelines for *M.tuberculosis* exposure control. As a respirator, it is intended to help reduce wearer exposure to certain airborne particles, including those generated by electrocautery, laser surgery, and other powered medical instruments. As a surgical mask, it is designed to be fluid resistant to splash and spatter of blood and other infectious materials<sup>‡</sup>.

### Contraindications:

3M recommends that this respirator is not for industrial use. Not for use with beards or other facial hair or conditions that prevent a good seal between the face and the sealing edge of the respirator. Does not protect against gases or vapors (i.e. anesthetic gases such as isoflurane or vapors from sterilants such as glutaraldehyde). **This respirator was not designed to be used by children.**

† Tested against a 0.3 micron particle (mass median aerodynamic diameter) per U.S. 42 CFR 84.
‡ Meets ASTM Fluid Resistant Challenge F1862.
± Meets ASTM Standard Test Method for Bacterial Filtration Efficiency F2101.

### Use Instructions:

1. In the U.S., before occupational use of this respirator, a written respiratory protection program must be implemented meeting all requirements of U.S. OSHA 29 CFR 1910.134 such as medical evaluation, training and fit testing. Select and use respirator in accordance with all applicable regulations, standards and professional guidance. Fit testing must be performed while the test subject is wearing any applicable safety equipment that may be worn during actual respirator use which could interfere with respirator fit, such as hair bonnets and eyewear. In Canada, CSA standard Z94.4 requirements must be met. Follow all applicable local regulations. This respirator is designed for occupational/professional use by adults who are properly trained in its use and limitations. The 3M FT-10 (sweet solution) or FT-30 (bitter solution) Qualitative Fit Test Apparatus or other OSHA accepted fit test protocols are recommended for fit testing this respirator.

- Inspect respirator before each use to ensure that it is in good operating condition. Examine all the respirator parts for signs of damage including the two headbands, staples, noseclip, nosefoam and facepiece material. Ensure there are no holes in the breathing zone other than the punctures around staples and no damage has occurred. Enlarged holes resulting from ripped or torn filter material around staple punctures are considered damaged. Staple perforations do not affect NIOSH approval. The respirator should be disposed of immediately upon observation of damage or missing parts.
- Leave the contaminated area immediately and contact your supervisor if dizziness, irritation or other distress occurs.
- Respirator may be used until damaged, breathing becomes difficult or contaminated with blood or body fluids. Discard after every use when used for surgical procedures. Follow national, state, local and facility infection control guidance and policies.

### Fitting Instructions: Must be followed each time respirator is worn.

- Cup the respirator in your hand, with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand.
- Position the respirator under your chin with the nosepiece up. Pull the top strap over your head resting it high at the top back of your head. Pull the bottom strap over your head and position it around the neck below the ears. Make certain hair, facial hair, jewelry and clothing are not between your face and the respirator as they will interfere with fit.
- Place your fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the nose area to the shape of your nose by pushing inward while moving your fingertips down both sides of the nosepiece.  
 Pinching the nosepiece using one hand may result in improper fit and less effective respirator performance (Use two hands).
- Perform a User Seal Check. To check the respirator-to-face seal, place both hands completely over the respirator and exhale. Be careful not to disturb the position of the respirator. If air leaks around nose, readjust the nosepiece as described in step 3. If air leaks around the respirator edges, adjust position of straps and make certain respirator edges fit snugly against the face. **If you CANNOT achieve a proper seal, DO NOT enter the contaminated area. See your supervisor.**

### Removal Instructions:

- Without touching the respirator, *slowly* lift the bottom strap from around your neck up and over your head. Then lift off the top strap. Store or discard according to your facility's infection control policy. Dispose of used product in accordance with applicable regulations.

### Storage Conditions and Shelf Life:

Before use, store respirators in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture and damaging chemicals. When stored in original packaging between temperatures from -4°F (-20°C) to +86°F (+30°C) and not exceeding 80% RH, the product may be used until the date specified on packaging located next to the "Use by Date" symbol.

	= Use by Date
	= Manufacturer
	= Date of Manufacture
<b>LOT</b>	= Manufacturer's Lot Number relevant to the device bearing the symbol

NIOSH Approved N95 Respirator:

At least 95% filtration efficiency against solid and liquid aerosols that do not contain oil.

	<b>3M</b> St. Paul, Minnesota, USA 1-800-243-4630 1860 Series Respirators				
THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:					
RESPIRATOR COMPONENTS					
FILTERING FACEPIECE					
TC-	PROTECTION <sup>1</sup>	1860	1860S	8612F	CAUTIONS AND LIMITATIONS <sup>2</sup>
84A-0006	N95	X	X		ABCJMNOP
84A-4526	N95			X	ABCJMNOP

- PROTECTION**  
**N95 - Particulate Filter** (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.
  - CAUTIONS AND LIMITATIONS**
    - A - Not for use in atmospheres containing less than 19.5 percent oxygen.
    - B - Not for use in atmospheres immediately dangerous to life or health.
    - C - Do not exceed maximum use concentrations established by regulatory standards.
    - J - Failure to properly use and maintain this product could result in injury or death.
    - M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
    - N - never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
    - O - Refer to User's Instructions, and / or maintenance manuals for information on use and maintenance of these respirators.
    - P - NIOSH does not evaluate respirators for use as surgical masks.
- Rev. F: 11-19-07

### FOR MORE INFORMATION

and assistance on 3M Health Care products contact your local 3M representative or call 3M Healthcare Helpline toll free in USA 1-800-228-3957. In Canada, call 3M Helpline at 1-800-563-2921.

Made in U.S.A. with U.S. and imported materials by

 **3M Health Care**

2510 Conway Ave.

St. Paul, MN 55144

1-800-228-3957

www.3M.com/healthcare

© 3M 2012. All rights reserved.

3M is a trademark of 3M Company, used under license in Canada.

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